



MAKERS OF RESPIRATORY VALVES SINCE 1938

HANS RUDOLPH, inc.

www.rudolphkc.com

TEL: (913) 422-7788

U.S.A. & CANADA: (800) 456-6695

FAX: (913) 422-3337

E-Mail: hri@rudolphkc.com

8325 COLE PARKWAY, SHAWNEE, KANSAS 66227 U.S.A.

Data Sheet...

7500 Series V2 Mask™

ORO-NASAL NIV - No AAV - No Vents REUSABLE

Large, Medium, Small, Extra Small, Petite

PRODUCT FEATURES

- **Five Sizes** to cover a wide range of patient sizes.
- **Face Piece with new Facial Sealing Flange** is a single piece mask, made of clear soft Silicone Rubber.
- **Anatomically Contoured Face Mask Design with ribbed support** for a leak free comfortable fit.
- **Elbow shaped or Straight Swivel Ports** in both 22mm outside diameter or 22mm inside diameter allow for improved patient freedom of movement. Color coded translucent green.
- **Easily adjustable, light weight, 4 strap Headgear** for patient comfort, available in three sizes.
- **New Quick-Release Strap Clips** for quick & simple mounting & dismounting of headgear.
- **Lightweight, easy to use, high quality mask & headgear.**
- **Reusable masks; steam autoclaveable, cold chemical, pasteurization safe & dishwasher safe.**

SPECIFICATIONS

MATERIALS (See Fig. A)	
Face Piece with Structural Braces	Silicone Rubber, Polycarbonate Thermoplastic
Swivel Port - Mask Adapter, Elbow, 22mm Port	Polycarbonate Thermoplastic, Translucent Green
Headgear	Polyurethane Foam Black, Nylon UBL Gray and Nylon Fabric Red
Headgear Hook	Nylon, Black
Headgear Strap Clips (4), Tri-glide	Polypropylene, Black




Oro-Nasal NIV V2 Mask™ with Headgear
Elbow and Straight Type Swivel Port Shown

GENERAL INFORMATION

- **Intended Use:** The 7500 Series V2 Masks™ are reusable, multi-patient-use, adult oro-nasal masks intended to provide a patient interface for application of noninvasive ventilation. The masks are to be used as an accessory to continuous ventilators which have adequate alarms and safety systems for ventilator failure.
- **Indications for Use & Environment:** The masks are specifically indicated for use on adult patients (greater than 30 kilograms weight) to administer positive pressure ventilation for treatment of respiratory failure or respiratory insufficiency. They are intended for use on patients who are appropriate candidates for noninvasive ventilation, in the home, hospital, or other clinical settings by individuals that have received at least minimal instruction or training on the use of the masks as well as the device and system to which the masks are intended to connect.
- **Cautions:**
 - a.) Federal law restricts this device to sale by or on the order of a physician.
 - b.) Patients with facial hair, especially beards, may experience mask leakage which may require shaving the hair.
- **Warnings:** Refer to: Instructions for Use, Document No. 691271, shipped with product.
- **Contraindications & Complications:** Refer to: Instructions for Use, Document No. 691271, shipped with product.

PHYSICAL CHARACTERISTICS



Face Mask Dimensions

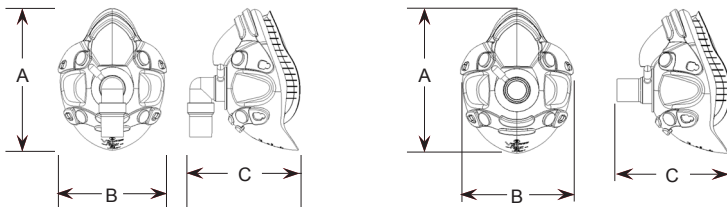


	MASK SIZE WITH OVERALL DIMENSIONS - inches (mm)					
	Elbow Swivel Port			Straight Swivel Port		
	A	B	C	A	B	C
Large	5.9 (150)	4.2 (107)	4.71 (120)	5.9 (150)	4.2 (107)	4.53 (115)
Medium	5.5 (140)	4.16 (105)	4.46 (113)	5.5 (140)	4.16 (105)	4.28 (109)
Small	5.2 (132)	4.05 (103)	4.31 (109)	5.2 (132)	4.05 (103)	4.12 (105)
X-Small	4.8 (122)	3.95 (100)	4.21 (107)	4.8 (122)	3.95 (100)	4.04 (103)
Petite	4.6 (117)	3.9 (99)	4.05 (103)	4.6 (117)	3.9 (99)	3.92 (100)

Mask Deadspace Volume (ml) With 22mm ID Elbow Swivel Port				
Large	Medium	Small	Extra Small	Petite
169	151	125	114	104

7500 V2 Mask™ COMPONENTS FIG. A

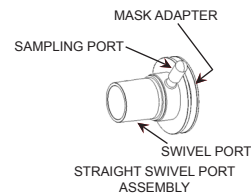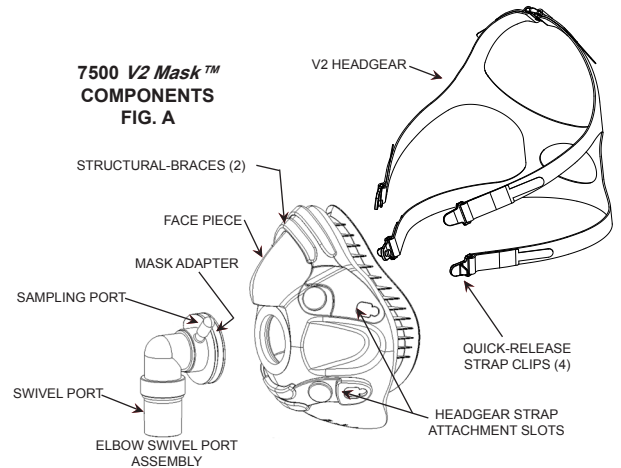

MASK SIZING GAUGE

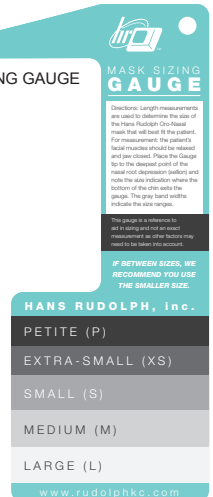

Commonly Ordered Items

Elbow Syle Swivel Ports

P/N	Description
	Mask, No AAV, Non-Vented 22mm ID Elbow Type with HG
113510	7500 Large
113511	7500 Medium
113512	7500 Small
113513	7500 Extra Small
113514	7500 Petite

Accessories

P/N	Description
691143	Mask Sizing Gauge for V2 Oro-Nasal Masks
669353	METRO™ NG Tube Seal Disposable Package of 1

Replacement Parts

P/N	Description
669180	Face Piece Large
669181	Face Piece Medium
669182	Face Piece Small
669183	Face Piece Extra Small
669184	Face Piece Petite
201494	Headgear Large (Use with Large Mask)
201493	Headgear Medium (Use with Medium and Small Mask)
201492	Headgear Small (Use with XSmall and Petite Mask)
201532	Swivel Port Elbow Type 22mm ID
201537	Swivel Port Straight Type 22mm ID
201531	Swivel Port Elbow Type 22mm OD
201536	Swivel Port Straight Type 22mm OD
201555	HG Strap Clips Package of four & Tri-glide
211022	Cap Plugs for sampling port package of two

Other Ordering Options Available But Uncommon

Elbow Style Swivel Ports

P/N	Description
	Mask, No AAV, Non-Vented 22mm OD Elbow Type with HG
113505	7500 Large
113506	7500 Medium
113507	7500 Small
113508	7500 Extra Small
113509	7500 Petite

Straight Style Swivel Ports

P/N	Description
	Mask, No AAV, Non-Vented 22mm ID Straight Type with HG
113520	7500 Large
113521	7500 Medium
113522	7500 Small
113523	7500 Extra Small
113524	7500 Petite

P/N	Description
	Mask, No AAV, Non-Vented 22mm OD Straight Type with HG
113515	7500 Large
113516	7500 Medium
113517	7500 Small
113518	7500 Extra Small
113519	7500 Petite

